# HeartGuard Polar -40°C

- **INSULATED STAINLESS STEEL CABINET**
- **FROST RESISTANT DOWN TO (-40 °C)**
- **ADDITIONAL FRONTAL HEATING ELEMENT (-20 °C TO -40 °C)**
- **FAN FOR VENTILATION**
- **2 X LED LIGHTS (MEANING DEFIBRILLATOR IS VISIBLE IN THE DARK)**
- **DESIGNED TO BE EASY TO OPEN, EVEN UNDER CONDITIONS OF HEAVY SNOW AND FROST.**
- **FUTURE-PROOF! (SOFTWARE CAN BE UPGRADED)**

## FUNCTIONAL DESCRIPTION:

If the door of the cabinet is opened:

The SIREN immediately sounds.  
(If the door of the cabinet is closed once again within 3 minutes the siren will stop automatically).

Once the door is closed:

The cabinet is once again ready for operation.

## TECHNICAL DESCRIPTION:

Door:

The door is fitted with thermopane glass and marked with international AED labels/symbols.

Light:

2 x LED lights in the cabinet, such that the defibrillator is visible in the dark.

Siren:

Siren in the cabinet. (100 dBA / 1m)

Fan:

Ventilation fan, starts automatically when temperature exceeds +40 °C.

Frost-proof cabinet:

Thermostatically-controlled heating element. Frost resistant down to -40 °C

Voltage:

Must be connected to a 230V (116Wh) electrical supply.

Overvoltage category:

Outdoor cable (3 x 1.5 mm<sup>2</sup>)

Protection rating:

Cat II, IE IEC 61010-1

Ambient temperature:

IP43.

Relative humidity:

-40 / +40°C

Environmental pollution degree:

0 – 95 %

Altitude:

2

Design norm:

Max. 2000 masl. (meters above sea level)

Insulation – fire class:

IEC61010 – with deviations in insulation fire class and service switch.

External dimensions:

E / F (EN13501)

Internal dimensions:

W x H x D = 44 x 46 x 27 cm (incl. door handles)

W x H x D = 33 x 38 x 20 cm

Weight:

Gross weight incl. packaging 15 kg.

## SELF-TEST:

To ensure that the fan and heating elements are working, it is recommended to perform a self-test 1-2 times a year.

The self-test starts when the power is turned ON or by pressing the service button once. Now a 'BEEP' will sound. The fan and heating element will run for 1 minute. A 'BEEP' will sound again and the test is over.

*(Feel under the bottom plate on the right side of the cabinet and on the inside front to see if the heating elements are hot.*

*If you cannot feel the heat, repeat the test 2-3 times)*

## MAINTENANCE/CLEANING:

We recommend cleaning the outer surface of the cabinet at least 1

1-2 times per year.

Wipe using detergent and a damp cloth or brush and rinse with clean water. Finally wipe down the cabinet with a dry cloth.

If the surface of the cabinet becomes tarnished, this will be due to deposits which can easily be cleaned off.

For difficult deposits, use a scouring pad.

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